



Maintenance Frequency and Log

Pump Installed by: _____

Phone: _____

The purchaser has been informed of the process of maintenance and precautions of utilizing the equipment. Yes No

Signature: _____

Date: _____

Daily:

Clean Vacuum Piping System

Use non-foaming vacuum system cleaner

Weekly:

Clean Vacuum Pump Intake Filter(s)

Will be completed on _____ of every week.

Clean In-Operatory Strainer(s)

Will be completed on _____ of every week.

Semi-Annually:

Clean and Dust off Vacuum Pump

_____ Date _____ Date _____ Date _____ Date

_____ Date _____ Date _____ Date _____ Date _____ Date _____ Date

_____ Date _____ Date _____ Date _____ Date _____ Date _____ Date

Annually:

Clean or Replace Relief/Regulator Valve(s) and Filter(s)

_____ Date _____ Date _____ Date _____ Date _____ Date _____ Date _____ Date

Replace Recycler Return Line(s)

_____ Date _____ Date _____ Date _____ Date _____ Date _____ Date _____ Date

Clean Water Inlet Strainer

_____ Date _____ Date _____ Date _____ Date _____ Date _____ Date _____ Date

**Clean or replace Flow Regulator Strainer as necessary.*




Maintenance Instructions

A. Vacuum Piping System

Cleans daily. (See Maintenance Chart). The vacuum system should be cleaned using a non-foaming vacuum system cleaner.

B. Intake Filter

Clean Weekly. (See Maintenance Chart). Locate on intake manifold (Shown on picture).

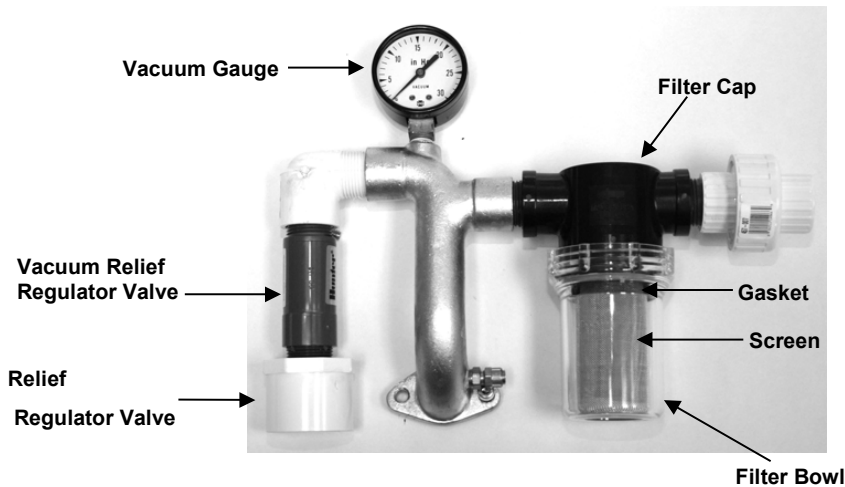

1. Turn pump "OFF".
2. Unscrew the filter bowl.
3. Remove screen and gasket. Clean both and lubricate gasket
4. Clean filter bowl and place gasket on sealing surface inside the bowl.
5. Fit the screen to the flange inside each bowl and screw the bowl to the cap. Hand tighten.
6. Turn pump "ON".

C. Operatory Strainer

Clean in-operatory strainers at least once a week.

D. Recycler Return Line

Replace annually (See Maintenance Chart). Locate line connected in front of each pump at recycler and intake manifold (Refer to the Vacuum Maintenance Chart). Use 1/2" open end wrench to remove and install polyflo.

E. Relief / Regulator Filters

Clean or replace annually. (See Maintenance Chart). Locate on intake manifold (Shown on picture). Unscrew filter from valve. Clean media or replace Relief / Regulator Filter.

F. Water Inlet Strainer

Clean annually. (See Maintenance Chart). Turn water "OFF". Use large crescent wrench to remove nut. Remove strainer and clean, then re-assemble.

G. Flow Regulator Strainer

Clean as needed (See Maintenance Chart). Located inside flow regulator.

JDS • 21205 Norwalk Blvd Hawaiian Gardens, CA 90716

Technical Support: (562) 924-4300 • Fax: (562) 924-2273 • Email: support@jdsmfg.com • Web: www.jdsmfg.com




Maintenance Instructions

A. Vacuum Piping System

Cleans daily. (See Maintenance Chart). The vacuum system should be cleaned using a non-foaming vacuum system cleaner.

B. Intake Filter

Clean Weekly. (See Maintenance Chart). Locate on intake manifold (Shown on picture).

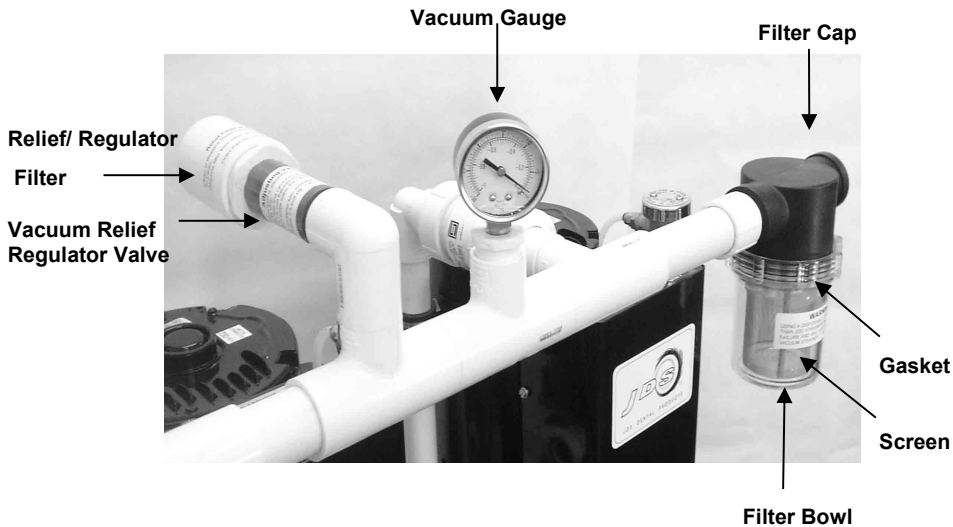

1. Turn pump "OFF".
2. Unscrew the filter bowl.
3. Remove screen and gasket. Clean both and lubricate gasket
4. Clean filter bowl and place gasket on sealing surface inside the bowl.
5. Fit the screen to the flange inside each bowl and screw the bowl to the cap. Hand tighten.
6. Turn pump "ON".

C. Operatory Strainer

Clean in-operatory strainers at least once a week.

D. Recycler Return Line

Replace annually (See Maintenance Chart). Locate line connected in front of each pump at recycler and intake manifold (Refer to the Vacuum Maintenance Chart). Use 1/2" open end wrench to remove and install polyflo.

E. Relief / Regulator Filters

Clean or replace annually. (See Maintenance Chart). Locate on intake manifold (Shown on picture). Unscrew filter from valve. Clean media or replace Relief / Regulator Filter.

F. Water Inlet Strainer

Clean annually. (See Maintenance Chart). Turn water "OFF". Use large crescent wrench to remove nut. Remove strainer and clean, then re-assemble.

G. Flow Regulator Strainer

Clean as needed. Located inside flow regulator (Recycler models only).

JDS • 21205 Norwalk Blvd. Hawaiian Gardens, CA 90716

Technical Support: (562) 924-4300 • Fax: (562) 924-2273 • Email: support@jdsmfg.com • Web: www.jdsmfg.com